MIJIN KORKA

AUTOMATIC BUBBLE SITZBATH

MJ-3000

MJ-2000

MJ-1000









¶ MJ-2000/3000 Functions & Advantages

Infrared Light Bath (MJ-3000 only) – 3 Level Amplifier

It allows you to warm up and strengthen blood circulation of affected area.

Double Filtering & Sterilizing

Tap water being filtered twice through 2 filters, it stores in the water tank. After then Ultraviolet Light Lamp built in the water tank sterilize it.

Ergonomic Design

It is ergonomically designed to be comfortable to sit on it and to maximize sitzbath-effect.

Air Bubble Effect

When taking sitz bath, your affected area can be washed and massaged by air bubbles generated under warm water.

Exchangeable Individual Bathtub

Exchangeable and individual bathtub prevents secondary infection inside of hospital / postpartum care center.

One–Touch, Full Automatic System

Touching start-button once, all functions work

- : 1st Operating (Feeding warm water sterilized by U.V lamp \rightarrow Sitzbathing with air bubbles \rightarrow Draining)
- → 2nd Operating (same to 1st operating)
- → Infrared light sitzbathing & Drying, Turning off

Safety Mechanism (Quadruple Safety Devices)

- -Temperature Controller
- -Maintaining the Right Temperature
- -Safety Fuse
- -Circuit Breaker

Self-Diagnosis Function

This product is composed of electronic circuits, so it self-diagnoses each function such as temperature. In the event of an abnormality, the safety shut-off system will be activated automatically. A specially designed intelligent program automatically diagnoses the fault area and produces a warning tone.

Emergency Stop Button

In an emergency, you can press the emergency stop button to stop all functions of the product.

MIJIN KOREA

PRODUCT SPECIFICATIONS

| MJ-2000/3000 |

Specifications

1. Water Supply Type: Water Pipe Connect 2. Sitz Bath Type: Warm water sterilized by UV & Automatic Air Bubble, Infrared Light Sitz-Bath

3. Water Tank Volume: 1.3 Liter 4. Warm Water Usage: 1.2 Liter

5. Warm Water Temp.: 38~42°C(Controllable)

6. Power Consumption: 1.8kW

7. Weight: 26 Kg

MJ-3000 only

8. Infrared Light: 150W(3 Level Amplifier) 9. Ultra Violet: UV-C(253.7nm), 4W

Consist of

1. Main Body - - - - - - - 1 Unit 2. Disposable sitz bathtub - 6 Unit

3. Fitting Tub(Water) - - - - 2 M

4. Drain Hose - - - - 1.5 M

Installation

1. Power: 220V, 60Hz(Wire Length 2m)

2. Power Consumption: 1.8kW

3. Water Pipe: 15mm

4. Bottom Drain Hole: 50Ø(Hose:35Ø)

5. Room Size: (W)1.0Mx(L)1.5 (min.)



| MJ-1000 |

Specifications

- 1. Water Supply: Direct supply of warm water
- 2. Sitz Bath Type: Automatic Air/Water Bubble
- 3. Air output pressure: 210 mmHg
- 4. Warm Water Temp.: within 42°C(recommend)
- 5. Bathtub storage capacity: 2 Liter
- 6. Power: 220V, 60Hz 10VA (adjustable)
- 7. Weight: 1.3 Kg

Consist of

1. Main Body 1 Unit
2. Bubbler 1 Unit
3. Backflow preventer 1 Unit
4. Transparent air hose 1.5 M

5 Safety plastic bag----- 1

6. User Manual - - - - 1





